

PEPPER MEDICAL**#5445****TUBE-TIE®****MULTI-PURPOSE ADHESIVE TUBE HOLDER**

with Quick Release non-slip Velcro® tab locking system

- NON-STERILE
- SINGLE USE
- DISPOSABLE
- LATEX-FREE

FOLEY CATHETER HOLDER:

Peel back base adhesive liner from Tube-Tie® securement device and apply to upper thigh. Open Quick Release Velcro® Locking Tab & peel off inner liner. Insert Foley Catheter below Y-lumen as pictured and engage Velcro® Tab. Velcro® Locking Tab may be lifted to reposition tubing.

**JEJUNOSTOMY & GASTROSTOMY TUBE HOLDER & WOUND DRAINAGE TUBE:**

Peel back base adhesive liner from Tube-Tie® securement device and apply next to insertion site. Open Velcro® Locking Tab & peel off adhesive inner liner. Coil the tubing comfortably and insert into locking system and engage Velcro® Tab.

SUBCLAVIAN LINE AND OTHER PORTS:

Apply Tube-Tie® next to site and use to prevent ports from dangling or dislodging.

Use Tube-Tie® to secure all types of tubing. Apply to clean, dry skin. Change patch and skin location if soiled or skin discomfort occurs. Remove securement device with alcohol or warm soapy water.

CONTRAINDICATIONS: DO NOT USE ON PATIENTS WITH KNOWN TAPE OR ADHESIVE ALLERGIES.



6 89466 24799 2

PEPPER MEDICAL INC.

www.peppermedical.com**(800) 647-0172**

REV 7-12

